

## **Transfer Cap YTRANS 2 for Pockets**

Transfer set adapted wallets and vials for reconstituting the drug.

### ***Description :***

This transfer set comprises of a stainless steel cannula for the reconstitution from a pocket and a drug vial with a diameter of 32mm.

Transfer Cap YTRANS 2

Ref: 500.362



# USE AND SAFETY

Read this manual before using this device.

It contains important safety information.

If you have questions, if you have any doubt, ask your doctor or pharmacist.

Keep this leaflet, you may need to refer to it again.

## Certificate **CE 0459**

### NAME AND ADDRESS OF THE OPERATOR

**P2A MEDICAL**

ZA de Montigny Les Vernes

### NAME AND ADDRESS OF MANUFACTURER

**P2A MEDICAL**

ZA de Montigny Les Vernes

## Transfer Cap Use




Step 1 - Identify the injection site, with the assistance of the arrow indicating the direction of flow from the outside of the bag.


Verify the compatibility of the drug and the solution.

Remove the lid of the capsule from the bottle and clean the stopper with antiseptic.

Step 2 - Remove the cap from its packaging.

After removing the protection film the needle, hold the transfer cap and push into the wide side of the bottle.

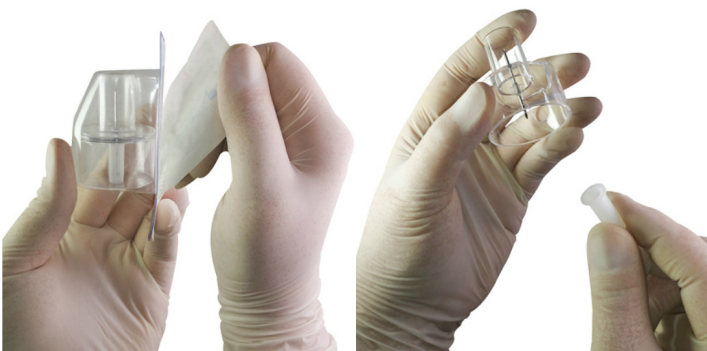

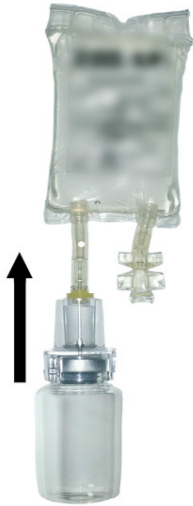

Step 3 - Perforate the sterile membrane by pushing in the needle.  
This allows the solution and drug concentrate to mix



Step 4 - Pass the solution into the bottle by squeezing the pocket.  
Shake well to mix.



Step 5 - In order to transfer the solution into the pocket.  
Turn the pouch into a downwards position.  
Then squeeze the pouch.  
Repeat the process if necessary.

Step 6 - When the recovery is complete, remove the entire  
transfer cap / vial of the pocket.

**THE TRANSFER HOOD IS FOR SINGLE USE ONLY.  
DISCARD ALL BOTTLES / TRANSFER CAPS IN A CONTAINER  
FOR SHARP OBJECTS.**