

Transfer Cap YTRANS for the Pocket

Transfer set adapted wallets and vials for reconstituting the drug.

Description :

This transfer set is in the form of a stainless steel cannula for the reconstitution from a pocket and a drug vial with a diameter of 22mm.

The hood has clips for an optimum grip on the neck of the bottle and a total security.

Transfer Cap YTRANS

Ref: 500.330



USE AND SAFETY

Read this manual before using this device.

It contains important safety information.

If you have questions, if you have any doubt, ask your doctor or pharmacist.

Keep this leaflet, you may need to refer to it again.

Certificate CE 0459

NAME AND ADDRESS OF THE OPERATOR

P2A MEDICAL

ZA de Montigny Les Vernes

74500 Maxilly sur Léman (FR)

NAME AND ADDRESS OF MANUFACTURER

P2A MEDICAL

ZA de Montigny Les Vernes

74500 Maxilly sur Léman (FR)

Transfer Cap Use



Step 1 - Identify the injection site, with the assistance of the arrow indicating the direction of flow from the outside of the bag.


Verify the compatibility of the drug and the solution.

Remove the lid of the capsule from the bottle and clean the stopper with antiseptic.



Step 2 - Hold the hood with its paper cover up and pull off the cover.

Keep the cap with its outer package and push the wide end at the top of the bottle.

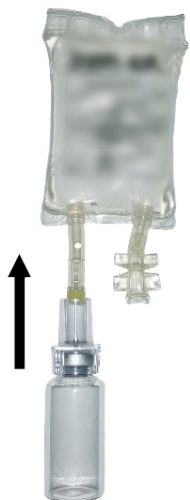

Step 3 - Perforate the sterile membrane by pushing in.

This allows contact between the solution and the concentrated drug to be diluted.



Step 4 - Pass the solution into the bottle by squeezing the pocket.

Shake well to mix.



Step 5 - In order to transfer the solution into the pocket Turn the pouch into a downwards position. Then squeeze the pouch.

Repeat the process if necessary.

Step 6 - When this process has been completed, remove the entire transfer cap / vial of the pocket.

THE TRANSFER DEVICE IS FOR SINGLE-USE ONLY.

THROW AWAY THE VIAL / TRANSFER DEVICE IN A SHARPS DISPOSAL CONTAINER.